



RFID

125KHz 134.2KHz 13.56MHz
 125KHz rfid 125khz rfid

- ◆
- ◆
- ◆
- ◆
- ◆
- ◆



□
□
R
E
D
□
□
□
□
□
□
□
□
□
□
□
□
E
M
4
B
5
/
N
T
A
G
2
1
6

MIFARE®DESFire®2K / EV1 2K / EV2 2K
MIFARE®DESFire®4K / EV1 4K / EV2 4K
MIFARE®DESFire®8K / EV1 8K / EV2 8K
MIFARE DESFire®NXP B.V.®

4. NFC®

1. NTAG®203 144 NTAG®213 144 NTAG®215 504 NTAG®216 888 NTAG®210 48
NTAG®212 128

NTAG®NXP B.V.®

2. MIFAREUltralight® 48 MIFAREUltralight®EV1 48 / 128 MIFAREUltralight®C 148
MIFARE®MIFARE Ultralight®NXP B.V.®

ISO 15693 / ISO 18000-3

ICODE®SLIX ICODE®SLIX-S ICODE®SLIX-L ICODE®SLIX 2

ICODE®NXP B.V.®



How Does Animal Microchip Work?



1. Scan the pet to verify whether he or she has a microchip that has been previously implanted.



2. Scan the microchip pack to verify that the microchip is functional, and that the ID code matches the accompanying barcode labels.



3. Implant the microchip, then rescan the pet to verify that the microchip can be localized.



4. Do not forget to record the tracking number of the microchip.

Implantation Sites for Microchips



Microchips are most often implanted in animals as outlined in the diagrams above. When scanning dogs and cats, begin in the neck area where the microchip is most likely to be located, but be sure to scan slowly and patiently. Scan repeatedly and over the entire body to ensure whether or not a microchip can be localized.



WHY YOU CHOOSE US AS YOUR PARTNER



NO. 1 MATERIAL

Material is medical grade bio-glass
with working life as long as 20 years

NO. 2 STERILIZATION

Sterilized by EO for 48 hours to
protect animal from being infected.

Safe enough to be injected
into animal body.



NO. 3 CHIP

Imported original chip. Long reading range and long working life



ISO11784/5, FDX-B, Chip Type: EM4305

ISO14443A/15693, Chip Type: NFC 216, etc

NO. 4 VARIOUS SIZES






NO. 5 **VARIOUS MODELS AND COLORS**

SYRINGES FOR ANIMAL ID TAGS



APPLICATION

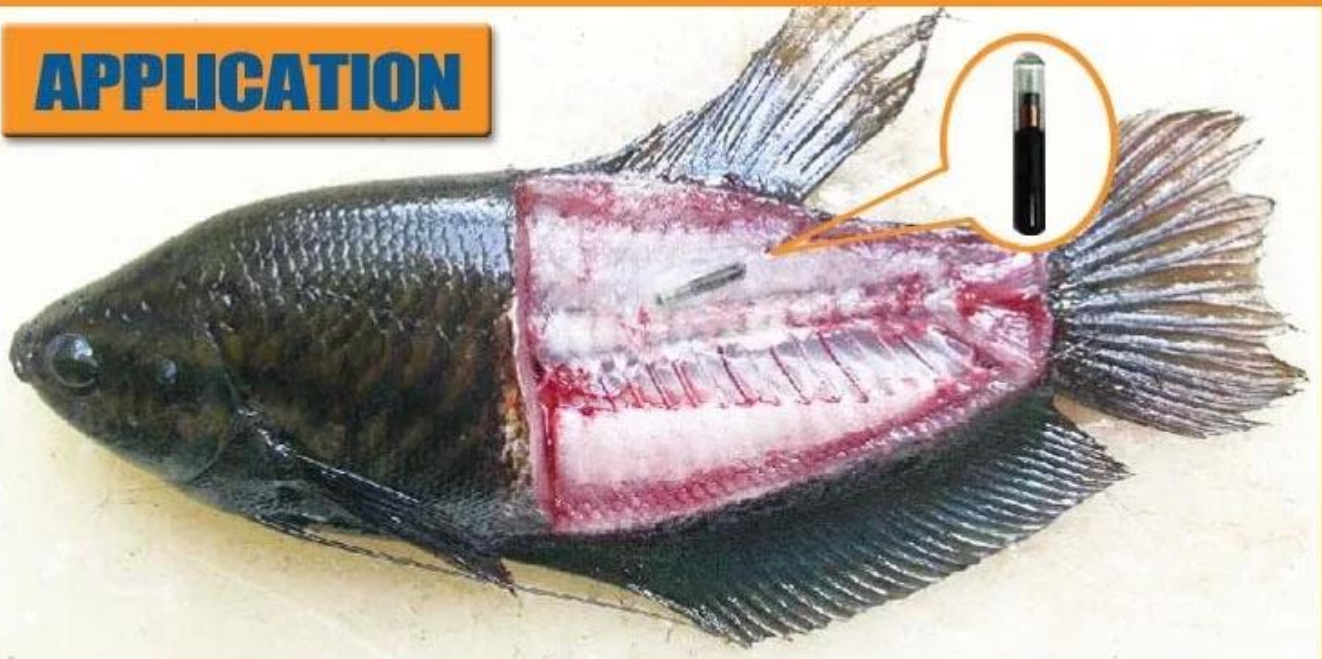

Fish tracking



Big Animal ID



Pet ID



Small animal ID



Bird ID



PACKING



1



2



3



4

包裝

- 1 包裝數量 24個
- 2 包裝數量可根據客戶需求調整
- 3 OEM / ODM 包裝
- 4 包裝上印有fashin包裝數量可根據客戶需求調整
- 5 包裝上印有
- 1 包裝數量可根據客戶需求調整
- 2 包裝數量可根據客戶需求調整
- 3 包裝數量可根據客戶需求調整2-3個
- 6 包裝數量可根據客戶需求調整15個或30個
- 7 包裝數量可根據客戶需求調整
- 8 包裝數量可根據客戶需求調整DHL、FEDEX、TNT、UPS、EMS、SEA、AIR

包裝

- 包裝數量可根據客戶需求調整ACM Goldbridge包裝數量2個
- 包裝數量可根據客戶需求調整ACM Goldbridge包裝數量
- 包裝數量可根據客戶需求調整

包裝

ACM Goldbridge

RFID 20 RFID RFID RFID RFID UHF



3 RFID 20 RFID



GOLDBRIDGE

405 518031
+ 0086-755-83767678
0086-755-88391348

金橋公司

微信/ whatsapp / 手机 +

0086-13554918707

Skype:lily-jiang1206

邮箱:sales@goldbridge sz.com

网站:[http:// www goldbridge sz.com](http://www.goldbridge sz.com)