



RFID Animal Glass Tag

rfid 125KHz 134.2KHz 13.56MHz

125KHz rfid 125khzrfid

- ◆
- ◆
- ◆
- ◆ LF HF
- ◆
- ◆



REF
D
E
M
A
B
E
N
T
A
G
2
1
6

A vertical thermometer with a scale from 0 to 100 degrees Celsius. The liquid level is at 50 degrees Celsius.

1. MIFARE Classic® 1K MIFARE Classic® EV11K MIFARE Classic® 4K
MIFARE MIFARE Classic NXP B.V.



How Does Animal Microchip Work?



1. Scan the pet to verify whether he or she has a microchip that has been previously implanted.



2. Scan the microchip pack to verify that the microchip is functional, and that the ID code matches the accompanying barcode labels.



3. Implant the microchip, then rescan the pet to verify that the microchip can be localized.



4. Do not forget to record the tracking number of the microchip.

Implantation Sites for Microchips



Microchips are most often implanted in animals as outlined in the diagrams above. When scanning dogs and cats, begin in the neck area where the microchip is most likely to be located, but be sure to scan slowly and patiently. Scan repeatedly and over the entire body to ensure whether or not a microchip can be localized.



WHY YOU CHOOSE US AS YOUR PARTNER



NO. 1 MATERIAL

Material is medical grade bio-glass
with working life as long as 20 years

NO. 2 STERILIZATION

Sterilized by EO for 48 hours to
protect animal from being infected.

Safe enough to be injected
into animal body.



NO. 3 CHIP

Imported original chip. Long reading range and long working life



ISO11784/5, FDX-B, Chip Type: EM4305

ISO14443A/15693, Chip Type: NFC 216, etc

NO. 4 VARIOUS SIZES






NO. 5 **VARIOUS MODELS AND COLORS**

SYRINGES FOR ANIMAL ID TAGS



APPLICATION

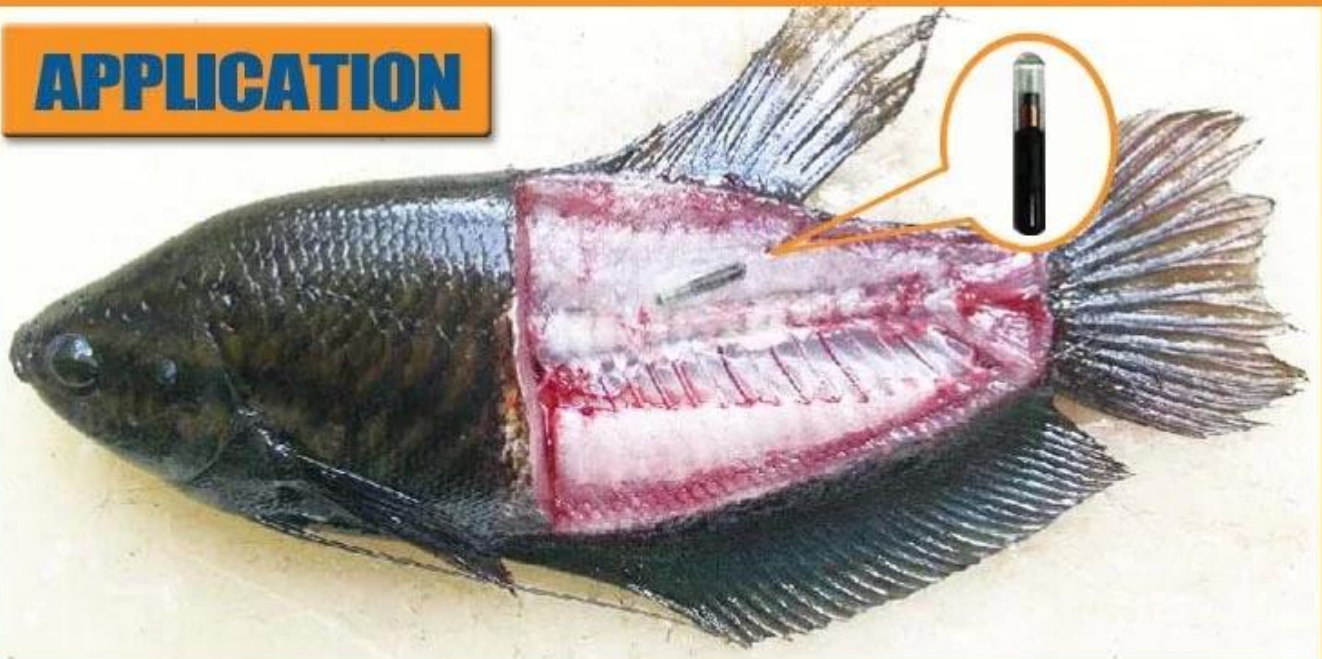

Fish tracking



Big Animal ID



Pet ID



Small animal ID



Bird ID



PACKING



服務項目

- 1 專業醫療器材包裝 24 小時
- 2 專業醫療器材包裝及運輸服務
- 3 OEM / ODM 服務
- 4 專業醫療器材包裝及運輸服務
- 5 專業醫療器材包裝
- 1 專業醫療器材包裝及運輸服務
- 2 專業醫療器材包裝及運輸服務
- 3 專業醫療器材包裝及運輸服務 2 3 專業醫療器材包裝
- 6 專業醫療器材包裝及運輸服務 1 5 專業醫療器材包裝 7 30
- 7 專業醫療器材包裝及運輸服務 T / T 專業醫療器材包裝及運輸服務
- 8 DHL FEDEX TNT UPS EMS SEA AIR 專業醫療器材包裝及運輸服務

服務項目

- 專業醫療器材包裝及運輸服務 ACM Goldbridge 專業醫療器材包裝 2 專業醫療器材包裝
- 專業醫療器材包裝及運輸服務 ACM Goldbridge 專業醫療器材包裝及運輸服務
- 專業醫療器材包裝及運輸服務



The collage consists of nine images arranged in a grid-like fashion. The top row features three images: a worker operating a large industrial machine, a close-up of a worker's hands assembling a component, and a view of the factory's exterior building. The middle row contains three images: a worker operating a machine, a close-up of a worker's hands assembling a component, and a view of the factory's exterior building. The bottom row includes three images: a worker operating a machine, a close-up of a worker's hands assembling a component, and a view of the factory's exterior building.



**SHENZHEN GOLDBRIDGE
INDUSTRIAL CO., LTD**

□□□

□□405□□□□□□□□□□□□□□□□518031

Tel□+ 0086-755-83767678 Fax□+

0086-755-88391348

□□□□□□□□□□

Mob / whatsapp / wechat□+

0086-13554918707

Skype□lily-jiang1206

E□□□□sales□at□goldbridge sz.com

□□□□□□□http□// www□ goldbridge

sz.com